

Instructions for Use:

- 1) Open the Rotunda Umbifunnel pack.
- 2) Place the Umbifunnel and Umbistand on your workspace.
- 3) Open your blood tube and place the lid on the base of the Umbistand.
- 4) Place the blood tube into the Umbistand.
- 5) Place the Umbifunnel into the blood tube and *gently* push down to secure.
- 6) Place the cut end of the umbilical cord into the Umbifunnel.
- 7) Milk cord until the sample is obtained – Do not fill above the indicator line on the blood tube.
- 8) Close and then invert the tube slowly, turning two (2) to four (4) times to ensure mixing of sample and anti-coagulant.
- 9) Remove gloves and replace with new gloves.
- 10) Tie-off bag containing placenta and cord.
- 11) Dispose of the placenta and cord in the Special Clinical Waste bin .
- 12) Dispose of the Umbifunnel in the Clinical Waste bin.
- 13) Remove blood tube and screw on lid.
- 14) Remove Umbistand and packaging in the General Clinical Waste bin.
- 15) Dispose of gloves.
- 16) Label blood tube.
- 17) Place blood tube in the sample bag.

